



Test Name	Test Code	Collection Container	Patient Instructions	Pour-off Container(s)	Special Notes
<b>Urinalysis (UA):</b> UA, Complete UA, Macroscopic (dipstick) UA, w/Reflex to microscopic UA, Microscopic UA, Screen	5463 6448 7909 8563 7048		<b>Midstream</b> Allow a small amount of urine to pass into toilet. Midway through urination, place container in the flow of urine to catch a sample. Finish voiding in toilet.		Transfer urine from pediatric bags to a clean container, then to pour-off container, dispose of the bag while following hygiene guidelines.
<b>Urine Culture (UCx):</b> UCx (Routine) UCx (Comprehensive/Special)	395 3021		<b>Clean Catch Midstream</b> Wash hands. Use one (1) soap towelette, then three (3) saline towelettes. Follow midstream instructions listed above.		No additional tests can be conducted using this tube.
<b>Urinalysis AND Urine Culture:</b> UA, Complete w/ Reflex to Culture Culture & Sensitivity, if indicated UA, Complete w/Reflex to Culture (REFL)	3020 17667		<b>Clean Catch Midstream</b> Wash hands. Use one (1) soap towelette, then three (3) saline towelettes. Follow midstream instructions listed above.		For pediatric and renal abnormality patient samples, <u>under 5 mL</u> , please send a refrigerated specimen in a sterile cup labeled "Pediatric and Renal Abnormalities."
Chlamydia trachomatis RNA, TMA, Urogenital Neisseria gonorrhoeae RNA, TMA, Urogenital Chlamydia/N. Gonorrhoeae/T. Vaginalis RNA, Qualitative, TMA	11361 11362 11363 16492		<b>First-Catch</b> Should not urinate within 1 hour before collecting sample. Collect first passing of urine into container provided. 20-30 mls of urine is needed to test.		Using the provided pipette, add urine until the liquid level is <b>between the two black lines</b>
Albumin, Random Urine w/Creatinine   UDS-HOSPITAL   Electrophoresis and Total Protein, Random Urine   OSMOLALITY (U)   Protein, Total, Random Urine w/Creatinine   Protein, Total, 24-hr Urine w/Creatinine   hCG, Qualitative, Urine	6517 678 8459 1715 59084 757 8525 396		<b>Random Urine Collection</b> Hold container a few inches from urethra and urinate until container is half full. Finish voiding in toilet. Replace lid and screw tightly.		*Keep in container
Sexually Transmitted Infection Panel, QL, PCR Vaginosis/Vaginitis Panel, QL, PCR Candidiasis Panel, Qual, PCR Women's Health Panel, Qual, PCR	97641 97712 97642 97711		<b>Random Urine Collection</b> Hold container a few inches from urethra and urinate until container is half full. Finish voiding in toilet. Replace lid and screw tightly.		*Keep in container

**\* PLEASE NOTE: Urine samples sent to the laboratory that are more than 5 mL and not in the correct pour-off container will be rejected, leading to a recollection error. \***

\*Draw a line on collection container and instruct patient to notify if unable to provide quantity.

\*\*DLO accepts refrigerated specimen for UA on pediatric patients